HANDLEIDING FLEXINEB

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Inhoud Flexineb

1. Masker
2. Controller
3. Halsterstraps
4. Kabel
5. Inlaatklep
6. Vernevelbuis
7. Uitlaatklep
8. Medicatiecup
9. Oplader
10. MDI-Adapter
11. Extensie Cup
12. Zoutoplossing
13. Gedestilleerd water
14. Strap vernevelbuis
15. Transportbox

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Safety information for first use

Read the entire manual carefully before use. When using the Flexineb for the first time, remember that this is also the horse's first time, so be careful.

Practice putting on the mask first. Start by opening the exit valve. This way the horse can breathe normally. Gently place the Flexineb over the horse's nose so that it fits to the head. When the horse breathes normally and is used to the Flexineb, close the exit valve and the treatment can be started.

If the horse resists the mask, be patient. Let the horse calmly look at and smell the mask before attempting to fit the mask again.

If the horse begins to become frightened during treatment, open the exit valve to allow more air to be inhaled. Close the valve again as soon as the horse is relaxed.

In an emergency situation, the exit valve at the bottom of the mask can be pulled out to allow the horse to breathe freely.

Although the mask is soft and flexible, the aerosol chamber at the front and the controller are hard plastic, so be aware of the risks when the horse shakes its head with the Flexineb on the head.

MAKE IT A GOOD DAY

Open the exit valve.

Fill medication cup.

Connect cable.

Put on the mask.

Start nebulizing.

Roadmap

Preparation

1. Fully charge the controller before first use.
2. Check that the rubber inlet and exit valve can move freely.
3. Open the exit valve at the bottom of the mask to allow the horse to breathe freely and become familiar with the mask.
4. Fill the medication cup and then place it on the inlet valve.
The medication cup should never be filled on the inlet valve. Closing the medication cup can damage the valve.
5. Connect the cable from the controller to the medication cup.
6. The mask is now ready for use.

Start nebulizing

7. Hang the Flexineb on and make sure that the mask fits well around the nose.
8. When the horse is used to the mask, close the ventilation opening at the bottom, this prevents the aerosol from being lost through the exit valve.
9. Set the controller at the appropriate level. When switched on, the lamp of level 1 (slow nebulizing) flashes. By pressing again immediately you can switch between the different levels 1, 2 or 3. As soon as the lamp lights up permanently, the level is activated and the nebulizing starts.
10. The Aerosol chamber fills with aerosol and deflates when the horse inhales.
11. When the liquid in the medication cup has been completely atomized, switch off the controller with the on/off button.

Note well: The controller switches off automatically after 35 minutes. If the medication cup is not empty yet, restart the controller.

Scan the QR code for instructions or visit flexineb.nl for more information.

Choice of medication cup and level per liquid

There are three types of Medication Cups: grey, blue and green.

- **Gray medication cup (standard)** for airway maintenance. Nebulize prescription medication or liquids such as 0.9% saline, SilvaPlex, and ImmunoSan.
Level: 1 or 2 for volumes up to 10 mL. Level 3 for thicker liquids or with the extension cup, 30 mL.
- **Blue medication cup** for therapeutically prescribed medication, where maximum efficacy is very important.
Level: 3
- **Green medication cup** for difficult to nebulize liquids, which at level 1 or 2 do not nebulize within 15 to 20 minutes in the standard gray medication cup.
Level: 2 or 3 depending on volume and liquid.

Notes:

- If a medication or liquid is difficult to nebulize, dilute with 0.9% saline.
- Avoid prescription formulations containing polyethylene glycol or polypropylene glycol; these excipients may reduce the effectiveness of the products.
- Longer nebulization times can improve the delivery of medication to the lungs.
- Improper use of a green medication cup can lead to pooling at the bottom of the nebulizer tube (a little moisture at the bottom of the tube is normal).

Extension Cup, 30 ml

If you want to use the Extension Cup to nebulize up to 30 mL, the pressure and tension on the diaphragm can cause the inhalation to take longer than when nebulizing 3 x 10 mL.

Using MDI-adapter

Remove the controller, cable and medication cup. Insert the supplied adapter into the inlet on top of the aerosol chamber. Flip the Aerosol Chamber horizontally. Insert the inhaler and administer medication according to veterinary advice.

On the left the different levels, on the right the battery status.

Clean the medication cup after each use.

Set controller level

- LED blinks green: Controller is starting up. By pressing again immediately you can switch between the different levels 1, 2 or 3.
- LED lights up green: desired level is selected, the medication cup is filled and nebulization starts. Pressing again will turn off the controller.

●	Level 3:	Top LED	Fast nebulization
●	Level 2:	Middle LED	Standard nebulization
●	Level 1:	Bottom LED	Slow nebulization

LED blinks green:

- The medication cup is empty.
The controller will turn off after some time.

LED lights up red. This can mean:

- Broken connection
- Broken cable
- Defective medication cup

Battery status

LED lights up green:

- Top: battery is full
- Middle: battery is half full
- Bottom: low state of charge

LED lights up red:

- Bottom: low battery
- Bottom flashes: controller is shutting down, charge the battery.

Charge battery

The LEDs flash alternately during charging.

When the battery is fully charged, the top LED lights up green.

Cleaning Flexineb

Medication cup

1. Place the hard white cap on the medication cup to keep the electronic connection free of moisture.
2. Place one drop of Dreft dishwashing liquid in the cup and fill with distilled water. Close the blue cap and let the cleaner work for at least 5 seconds.
3. Shake the medication cup briefly and press the blue cap once to force the liquid through the gold membrane. Pour out the contents.
4. Rinse; fill the cup with distilled water. Close the blue cap, shake briefly and discard the contents.
5. Fill the cup with a little distilled water, close the blue cap and turn on the controller for about 10 seconds. This procedure removes any residue and cleans the membrane in the medication cup.
6. Let the cup dry with the blue cap open.

CAUTION: The gold-coloured membrane is very fragile, so do not touch it! Never immerse the medication cup in water or chemical solutions. This can permanently damage the diaphragm or reduce its life.

Mask & aerosol chamber

Wipe the mask and nebulizer chamber with a damp cloth. If the mask is very dirty, wash the bare mask and the nebulizer tube in lukewarm soapy water.

Controller

Only charge the battery when the LED flashes red, this way the battery will last the longest. In addition, it is important to use the electronic controller in dry conditions. Store the electronic components and medication cup dry and free of dust. Keep frost-free.

CAUTION: Never immerse the controller in water or any other liquid, as this will permanently damage the electronics. Do not use the Flexineb in the rain.

Customer service

We are happy to help you by e-mail. Send us your question and we will get back to you. Email: contact@flexineb.nl

Facebook: fb.me/flexinebnederland
 Instagram: [@flexinebnederland](https://www.instagram.com/flexinebnederland)
 Youtube: [Flexineb Nederland & België](https://www.youtube.com/FlexinebNederland)

Nortev LTD, Galway, Ireland

Flexineb is made in Ireland

www.flexineb.nl
www.flexineb.com